



# Acumed Temporal Bone Lab Drill Angle & Straight Handpiece Instructions on Set Up and Use

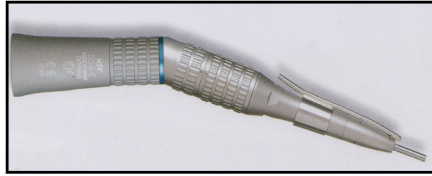

SGA-E Surgical Angle Handpiece

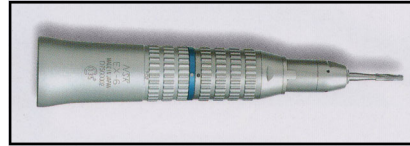

SGS-E Surgical Straight Handpiece

## Instructions for Surgical Angled and Straight Handpiece

### 1. DO'S and DONT'S

Do not operate handpiece without bur securely locked.

Do not operate handpiece when lock is open.

Never immerse in any solution.

Replace bur blank after use.

### 2. Recommended Burs

The Straight Handpiece accepts standard 3/32" (2.35mm) straight shaft burs, and burs should not exceed 2 3/4" (70mm) in length.

### 3. Attach Handpiece to Motor


Slide handpiece onto coupling of motor.

To remove handpiece, press button on Volvere motor and pull handpiece from motor. For other e-type motors, slide

### 4. Changing Burs

**To remove or insert a bur**, open the collet or locking ring in the direction of the black arrow until it clicks to stop.

**To seat a bur**, insert the bur all the way into the handpiece. Turn the locking ring in the direction of the white arrow until it clicks to stop.

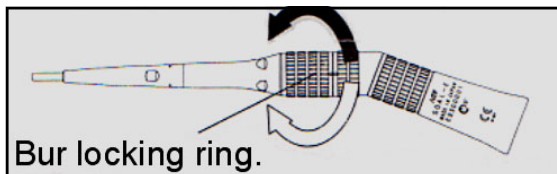

### 5. Lubrication

If handpiece is steam sterilized, use PANA-SPRAY before autoclaving.

Push E-type nozzle attachment over PANA-SPRAY nozzle until it firmly seats.

Shake can 3-4 times to mix lubricant.

Insert the E-type spray nozzle in the rear of handpiece and spray for 2-3 seconds.



**For additional information, please contact Acumed Instruments Corporation  
Phone: 1 800 234-5045 / Address: 5286 Evanwood Ave. Oak Park, CA 91377  
Website: [www.e-acumed.com](http://www.e-acumed.com) / E-mail: [acumed@earthlink.net](mailto:acumed@earthlink.net)**